


BIOCREDIT COVID-19 Ag

SARS-CoV 2 Antigen test

- » The first commercialization Black Gold Particle technology In the world (high sensitivity and specificity)
- » User friendly Dual Color System (Control line: Red, Test line: Black)
- » Rapid Test Time (5~8 Min)
- » Simple detection of SARS-CoV2 antigen

Core Technology Black Gold Particle

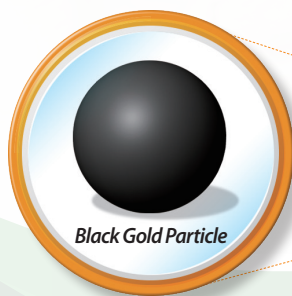


Differentiated Black Gold Conjugate Technology



Dual Color System

8 minutes is ok!

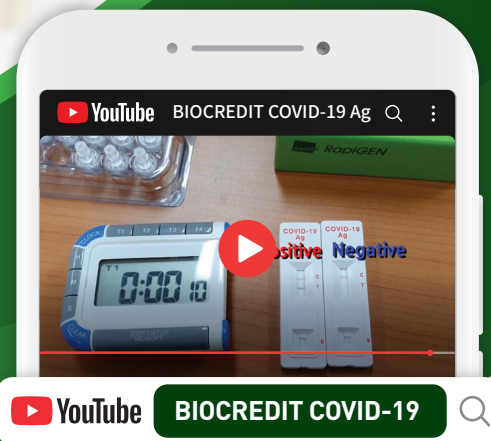

BIOCREDIT COVID-19 Ag rapid test kit

Intended Use	Detection of SARS-CoV2 antigen
Package	20 Tests / kit
Storage	2~30 °C
Specimen Type	Nasopharynx / Nasopharyngeal
Shelf life	24 months from manufacture date
Time to result	5~8 minute

COVID-19 Rapid Diagnostic Test

PROCEDURE


Insert the swab specimen and swirl the swab 5-10 times.

Remove the swab while gently squeezing the head of the swab.

Close the assay diluent tube with a filter cap securely.

Invert the assay diluent tube and gently squeeze it to draw 3-4 drops (90~150µl) into a sample well on the device.

Read the result within 5-8 minutes.

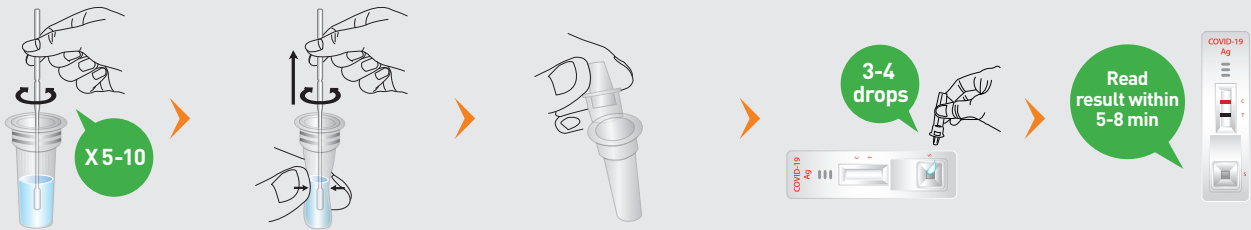

INTERPRETATION

NEGATIVE




One **red** line "C" within the result window.

POSITIVE



Two bands, **black** To test line and **red** "C" control line within the result window.

INVALID

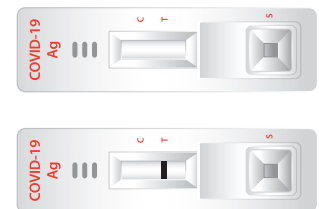

No "C" line within the result window. It is recommended that the specimen be retested.

